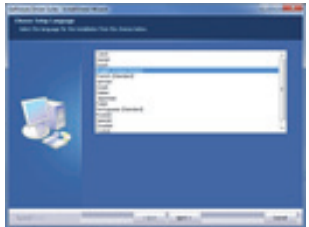
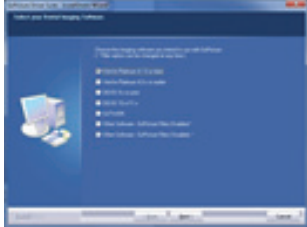
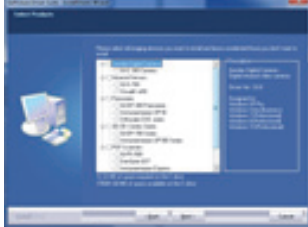
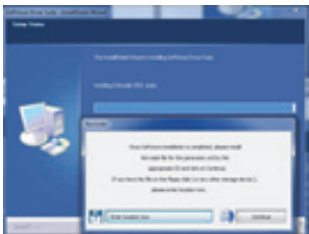
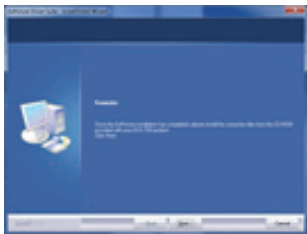
Quick Installation Guide for Gendex Software Driver (GxPicture™)

IMPORTANT: Before installing the GxPicture software driver, ensure the following:

- Temporarily disable any software, including antivirus software that may be running and may restrict access to computer resources (e.g. USB ports).
- Do not plug in any KaVo/Gendex digital device.
- For installation, you must login using an MS Windows system administrator account.
- VixWin Platinum users only: Upgrade VixWin Platinum software to the latest compatible version.
- Have installed all the latest service packs required by your Windows Operating System.

To install the GxPicture driver software either as a new installation or as an upgrade, do the following:

	<p>1. Insert the GxPicture Driver CD into the CD-ROM drive. <i>If the AutoPlay menu does not appear after a few moments, click the Start button, then select Run, and type D:\REV#GxPicture.exe (where D is the letter of the CD drive and REV# is GxPicture revision number). If the CD drive letter is not known, double click 'My computer' icon on the desktop, and make a note of the letter of the CD drive icon</i></p>		
<p>2. From the "Choose Setup Language" menu select the language for the installation and click Next.</p> 	<p>3. Select your Dental Imaging Software and click Next. Then click Install on the next screen.</p> 	<p>4. From the "Welcome" menu click Next. Read the "Software License Agreement" and click Yes to accept.</p> 	<p>5. Select the products to install/upgrade and click Next. Then click Install on the next screen.</p> 
<p>6. When installing GxPicture for a panoramic unit, you may be asked to install the mask file.</p> 	<p>7. Installing the GXS-700 also prompts to install the correction files from a separate CD at the end of the installation.</p> 	<p>8. When installing the GXC-300 or GXS-700, a prompt to plug the device into a USB port is displayed.</p> 	<p>9. After all files are installed click Next to complete the installation and reboot.</p> 

NOTE: When installing an Orthoralix DDE unit, install the Digital Module Unit Configuration CD after GxPicture installation is complete.

When upgrading previous versions of GxPicture, all existing KaVo/Gendex drivers are automatically detected and upgraded to the newest version. To add new devices, insert the GxPicture CD again after the version upgrade.

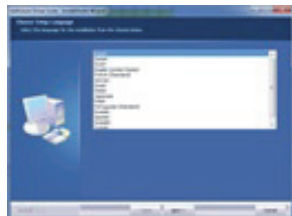
1910 North Penn Road
 Hatfield, PA 19440 USA
 Customer Service: 1-800-323-8029
 Technical Support: 1-800-769-2909
 Fax 1-877-292-6050
 www.gendex.com

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