User Manual

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Manuale d'uso

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Imaging Software

GENDEX™
Always by your side

Imaging Software

Introduction

This User Manual contains original instructions by Gendex Dental Systems. This manual describes how to acquire and process dental images by any TWAIN-compatible imaging software (such as practice management software or third-party imaging software) using GxTWAIN, the Gendex TWAIN interface.

The GxTWAIN interface is supported under the following operating systems:

- Windows 2000 Professional
- Windows XP Professional
- Windows Vista


Note: Please read this manual carefully before using the GxTWAIN interface.

Installation


Note: To install this software, you must have administrator rights on the computer. If you do not or have any problems, contact your system administrator.

To install GxTWAIN, perform the following steps:


1. Insert the GxPicture CD-ROM into the CD-ROM drive of your computer.
2. When the Welcome screen appears, click **Next** to continue.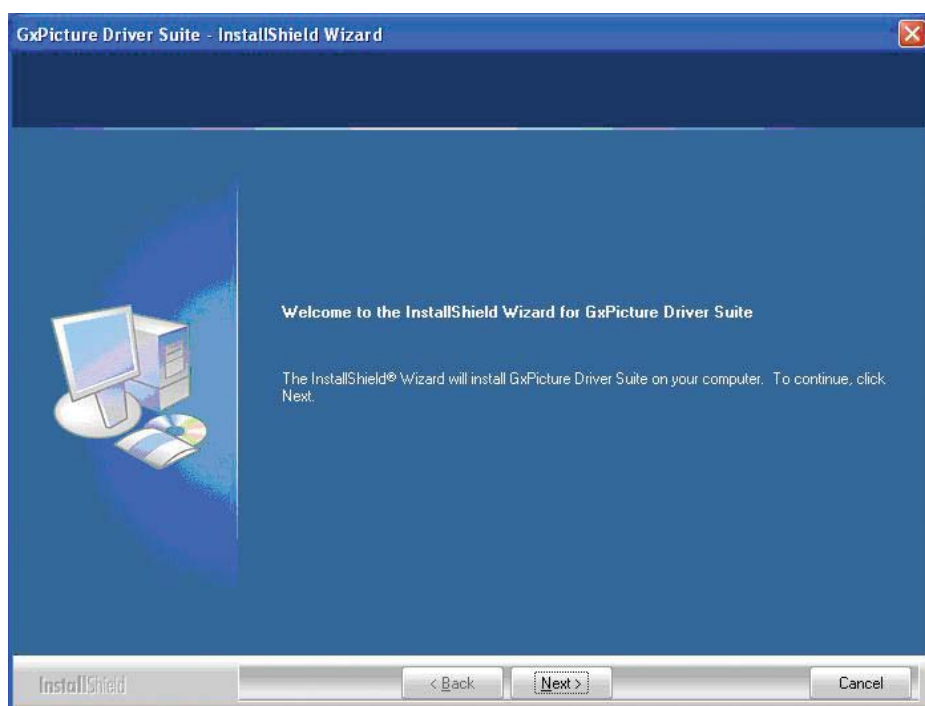


3. On the License Agreement screen, click **I accept...** and then click **Next**.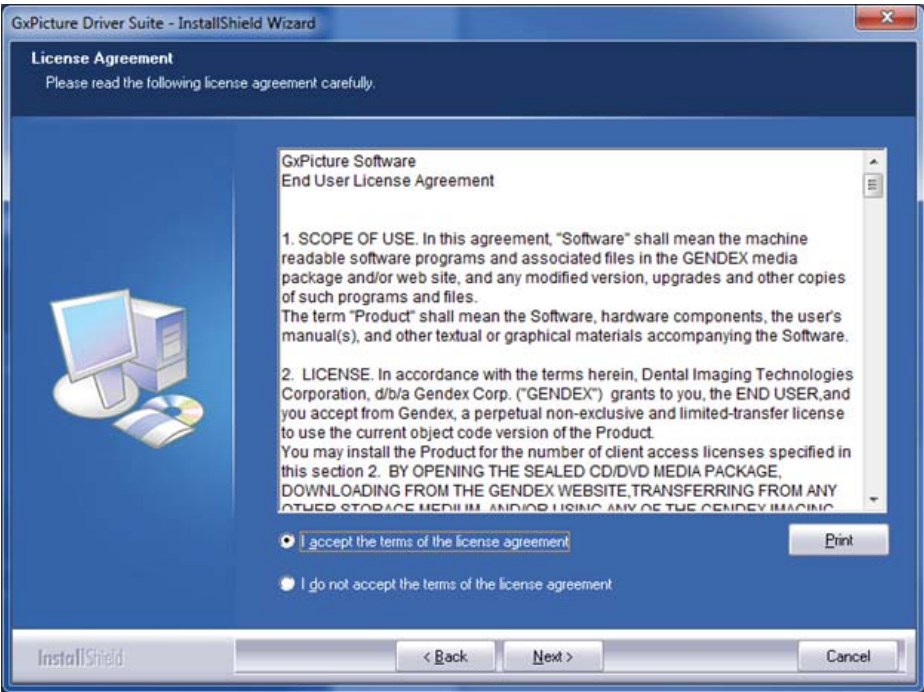


4. When the product list appears, select **GxTWAIN**, as well as any other imaging device you want to install, and then click **Next** to continue.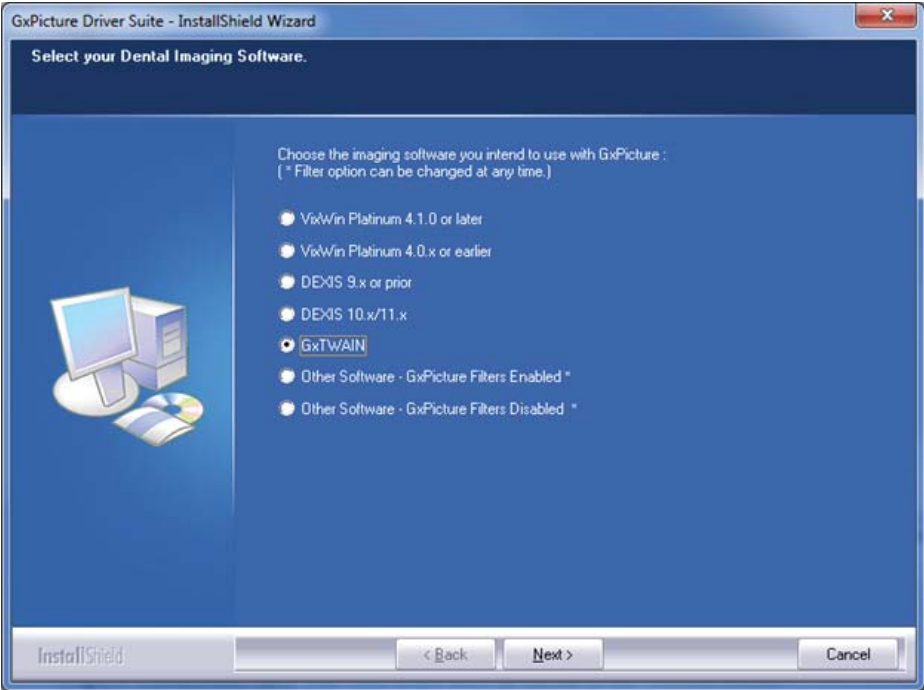

5. Click Install to begin the installation procedure.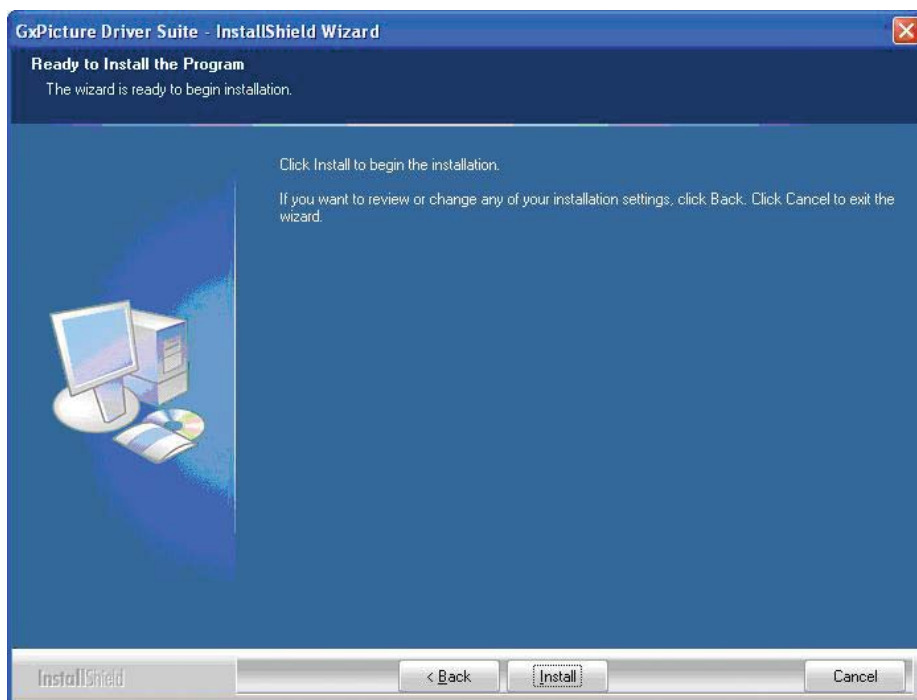

6. Click Finish to complete the installation procedure.

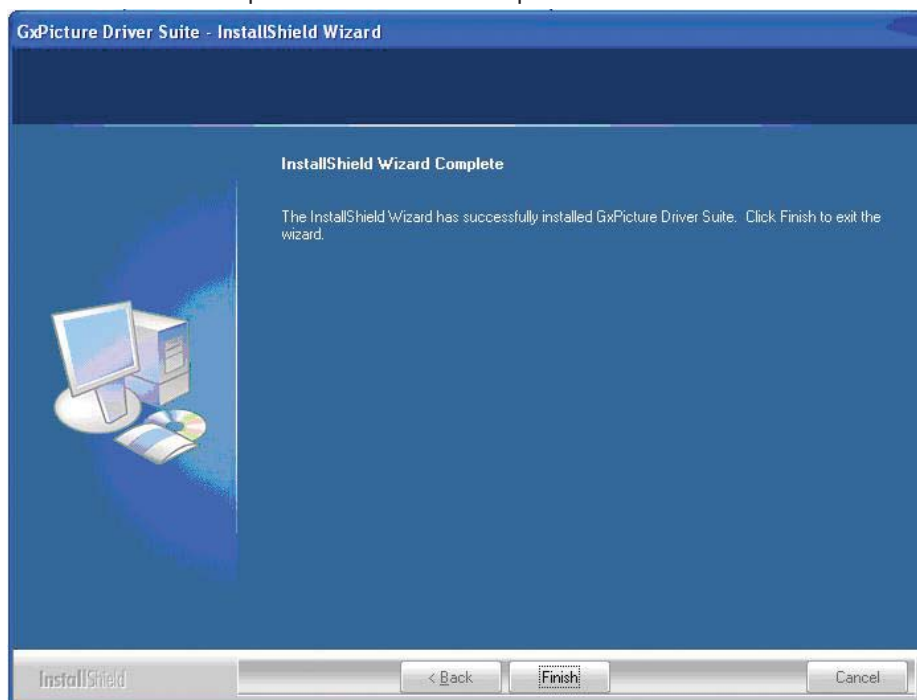
The installation procedure is now complete. If you want to use Gendex digital cameras from GxTWAIN, you must exit the GxVideo application and remove the shortcut according to the following steps:

- 1. Right-click the GxVideo camera icon in the system tray and select Exit.

- 2. Select Windows Start, All Programs, and then Startup. Right-click GxVideoApp and select Delete. When prompted, click Delete Shortcut.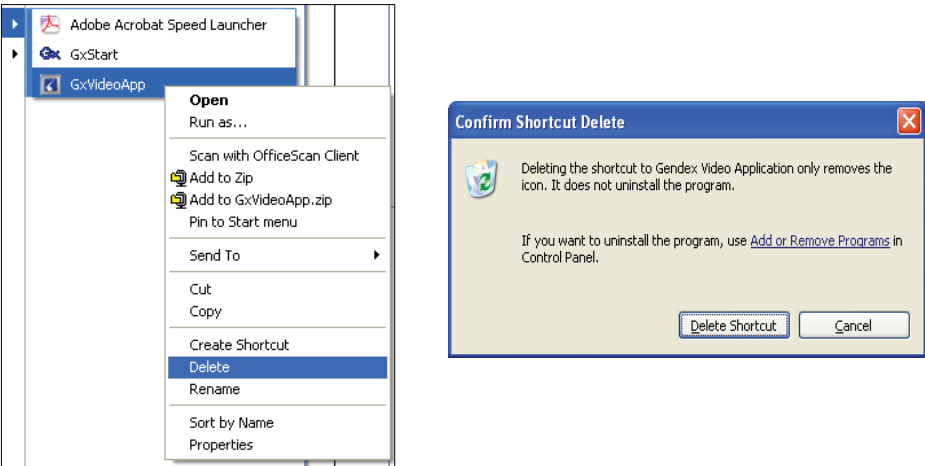



Before You Begin Working

Registration and Launching GxTWAIN


The system must be configured properly before launching GxTWAIN from your imaging application software.

You must register GxTWAIN before it can be used. Perform the following steps to complete the registration:

1. Locate the license number on the GxTWAIN CD-ROM sleeve and then go to www.gendex.com/vxregistration
2. Complete the online forms to receive a registration code.
3. Launch GxTWAIN. The procedure to acquire TWAIN images is different for every imaging program, but typically the option to acquire a TWAIN source is located within an Import option. After finding that, you will need to select **GxTWAIN** as the TWAIN source.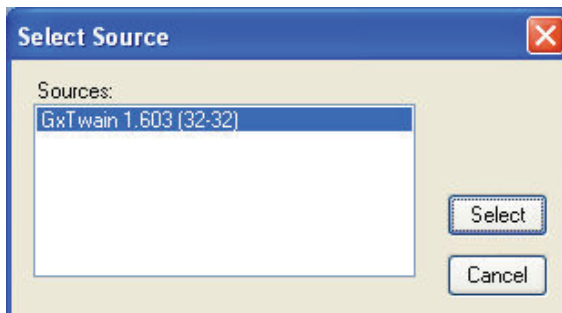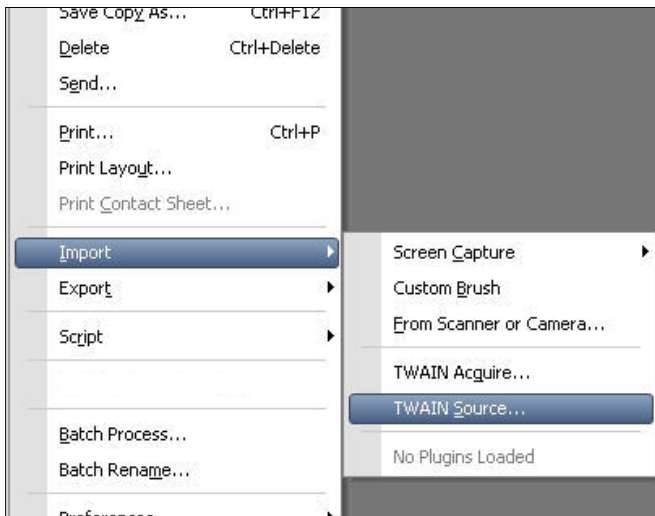

4. In GxTWAIN, click the **Registration** button near the bottom. Type in the Dental Practice Name, the Telephone Number, and the Registration Code. When finished, click **OK**.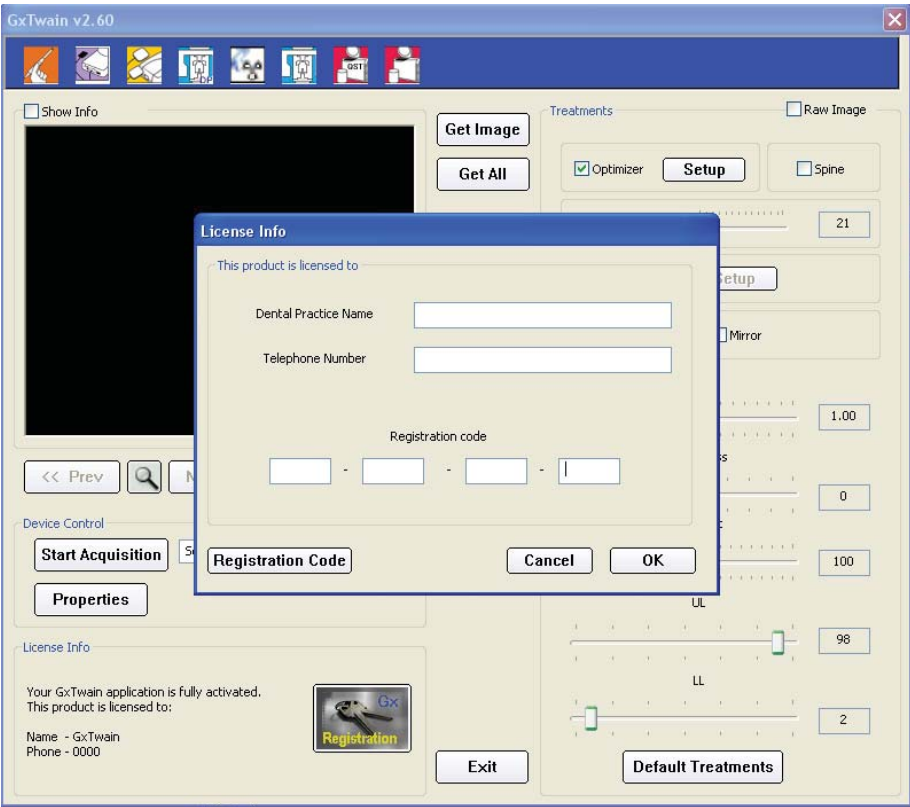


You can also register by phone:

U.S. and Canada	1-800-769-2909, ext. 3
Worldwide Technical Support	+49 7351 56 0

User Interface

The following graphic shows the various parts of the GxTWAIN main screen.

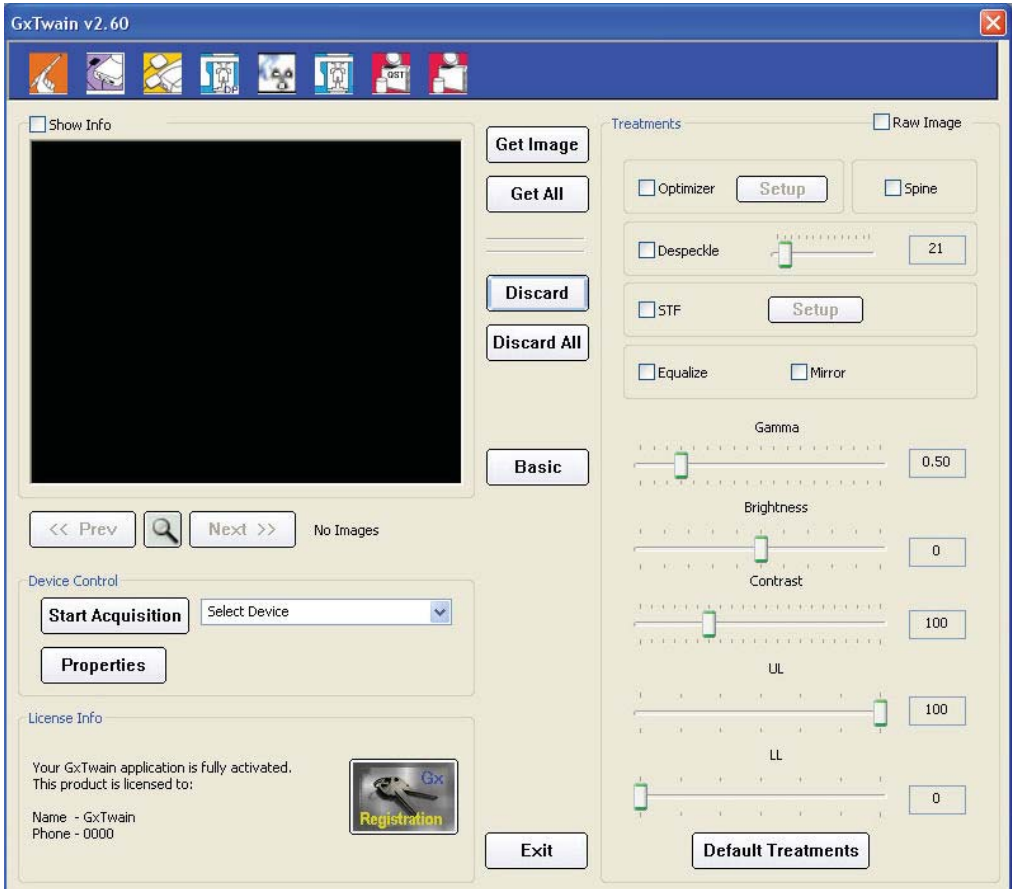
3.1 Installed Devices

The top left area of the screen shows all of the Gendex/Kavo imaging devices installed on the computer.


3.2 Image Area

When there is no image available, the Image Area will be dark.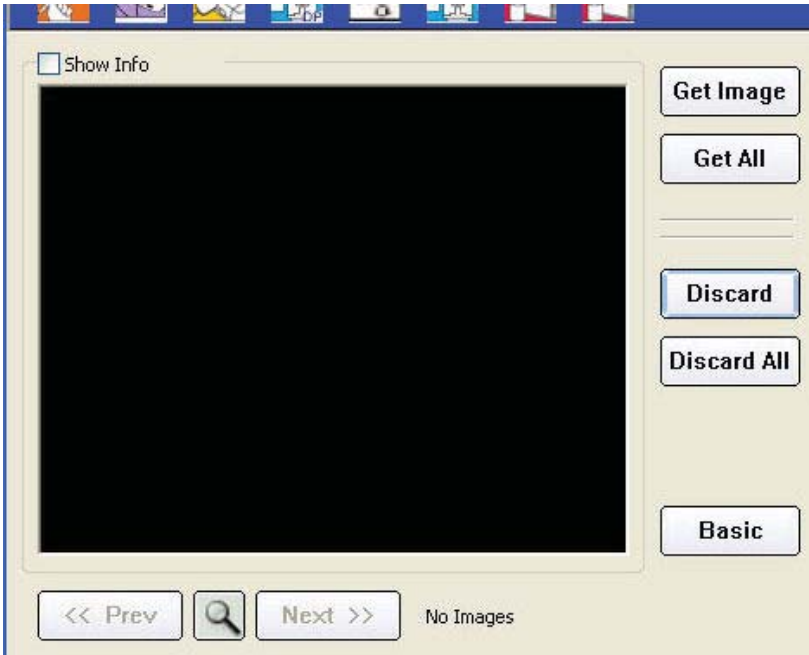

When an image or series of images is acquired, they will be previewed in the Image Area. The Show Info checkbox above the Image Area either enables or disables the display of information about the current image. The information will vary depending on the type of image being shown.

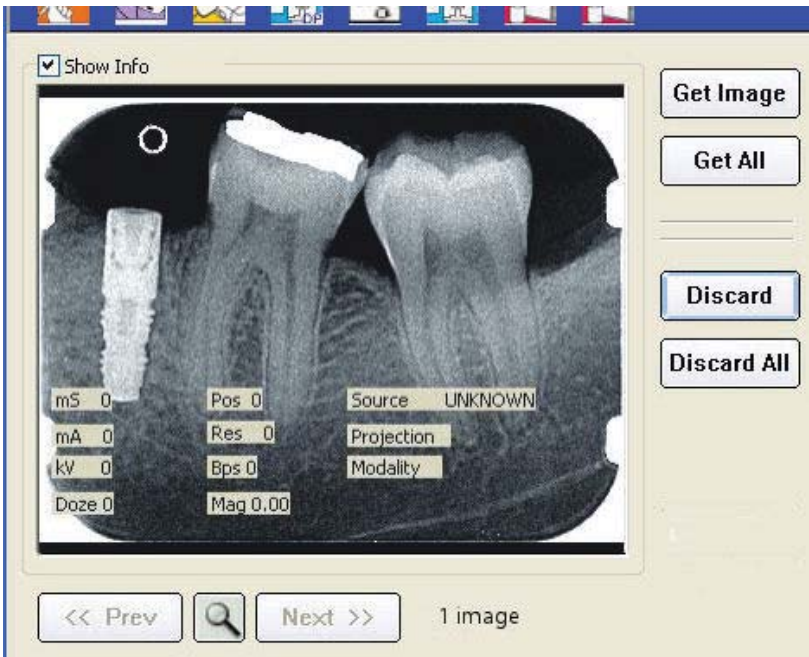You can magnify an image during a preview, as well as cycle forwards and backwards to the other available images. You can also either save or delete the images individually or as a group.

3.2.1 Image Area Buttons

A rectangular button with a light blue gradient and a thin black border. The text "Get Image" is centered in a bold, black, sans-serif font.

Click this button to transfer the current image to the imaging program on your computer.

A rectangular button with a light blue gradient and a thin black border. The text "Get All" is centered in a bold, black, sans-serif font.

Click this button to transfer all of the images in a set to the imaging program on your computer.

A rectangular button with a light blue gradient and a thin black border. The text "<< Prev" is centered in a bold, black, sans-serif font.

Click this button to display the previous image in the set.

Click this button to magnify the picture in the Image Area. (You can also double-click with your mouse in the Image Area to zoom an image.)

A rectangular button with a light blue gradient and a thin black border. The text "Next >>" is centered in a bold, black, sans-serif font.

Click this button to display the next image in the set.

A rectangular button with a light blue gradient and a thin black border. The text "Discard" is centered in a bold, black, sans-serif font.

Click this button to discard the current image.


A rectangular button with a light blue gradient and a thin black border. The text "Discard All" is centered in a bold, black, sans-serif font.

Click this button to discard all the images in the set.

3.3 Device Control

The Device Control buttons allow you to acquire images from a phosphor plate scanner (ex. DenOptix) or digital video cameras (ex. AcuCam CIV FWT).

The Properties button can be used to set device-specific settings, such as resolution, layout, etc.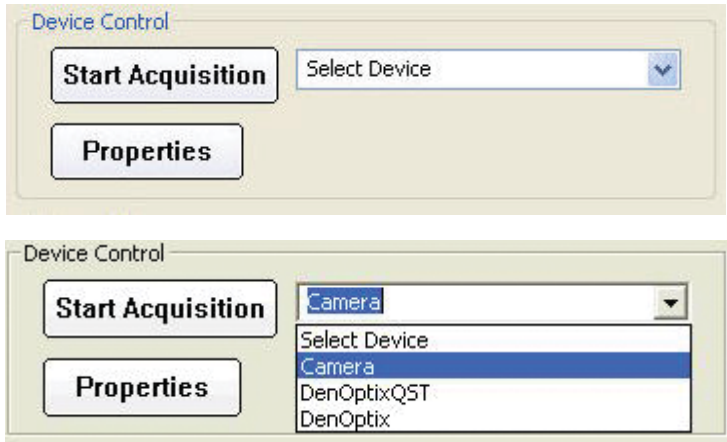

Note: To acquire images from direct digital devices, such as sensors (ex. Visualix) or pan/ceph systems (ex. Orthoralix 9200), no activation is required.

3.4 Treatments

The controls in the Treatments area are used to adjust a variety of image filter

Treatments ☐ Raw Image

Gamma

Brightness

Contrast

UL

LL

☒ Despeckle

☐ STF

☐ Equalize ☐ Mirror

values, such as Gamma, Brightness, Contrast, etc.

If you check the Raw Image box in the upper corner, all filters will be turned off. If you want to reset the all of the values to the originals, click the **Default Treatments** button.


3.4.1 Soft Tissue Filter (STF)

If you are using an Orthoralix 9200 DDE, you can use the Soft Tissue Filter option in the Treatments area. With this filter, you can optimize the display of soft and hard tissue in Latero-Lateral images.

You will need to select the STF checkbox and then click **Setup** to modify the settings.

In the STF Setup screen, you can set values for Bone, Soft Tissue, and Transition. After making any changes, you must click **Apply**. Click **Default** to return to the original settings.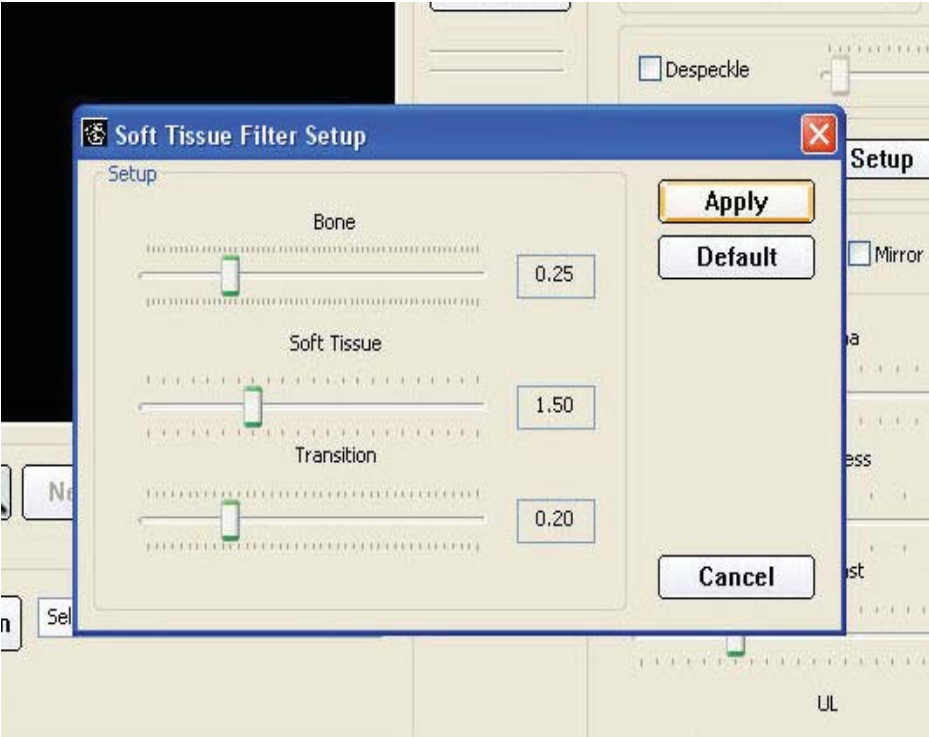

Acquiring Images

4.1 Using DenOptix or DenOptix QST


1. In the Device Control area, click the device selection pulldown and select DenOptix or DenOptix QST from the list of options.

2. If you need to make changes to the scan settings, click Properties. If you know the scan settings are set correctly, proceed to Step 4.

3. Make any necessary changes to the scan settings and then click OK.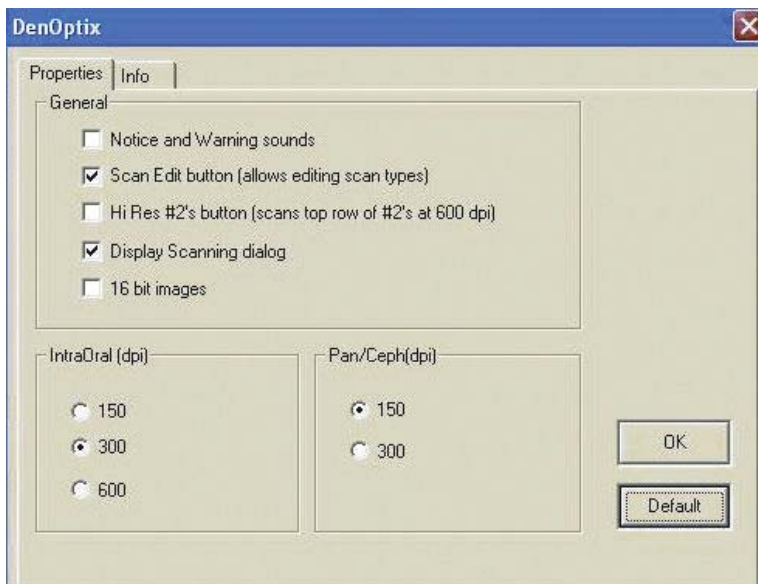


4. Click **Start Acquisition**. Scanned images will now appear in the Image Area.


4.2 Using Gendex Cameras

4.2.1 Initial Setup

- 1. In the main Device Control area, select **Camera** from the dropdown menu.
- 2. Select **Properties** to change any settings if this is either the first use of the camera or if any previously modified settings need to be changed.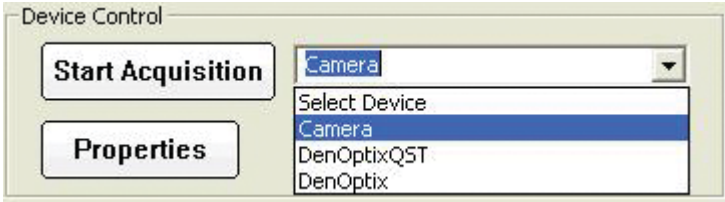

- 3. On the Properties window, click **Select** to choose a previously installed camera. After the camera is selected, you can modify other properties, such as Capture directory, Maintain aspect ratio, etc.

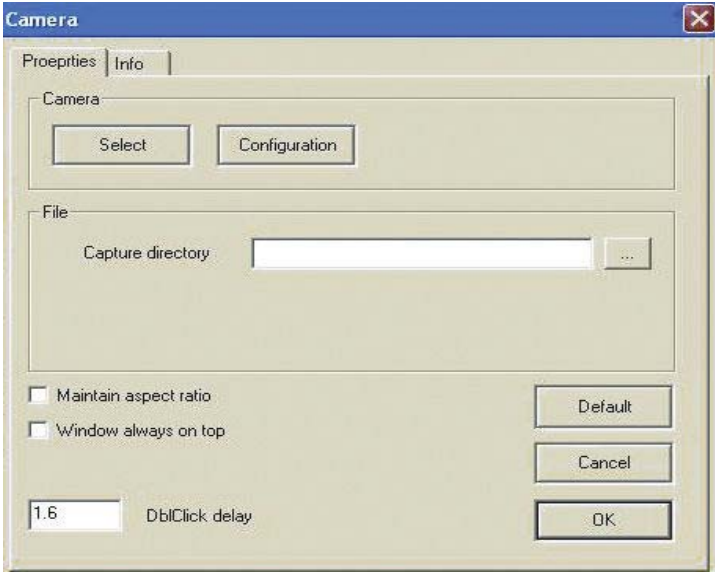Note: Click **Default** to use pre-defined values for these properties. In most cases, the default values should be sufficient, but you should check your system setup to figure out the best options that are required for your applications.

4.2.2 Start Acquisition

If this is the first time you are using GxTWAIN, make sure you have performed the steps in the previous section (Initial Setup).

1. Launch your imaging software.
2. Launch GxTWAIN from within your imaging software.
3. In the Device Control area, select **Camera** from the dropdown menu.
4. Click **Start Acquisition**.

Note: In most cases, the camera will begin taking images after you have clicked Start Acquisition. If the camera does not begin taking images, the camera may not be selected. If the camera is selected, you will see live video from which you can capture images. When the camera is returned to its stand, your live video should stop and the GxTWAIN main screen should appear.

4.2.3 Stop Acquisition


To exit video capture mode, either click **Stop Acquisition** or return the camera to its holder.

Recently captured images will appear in the Image Area of GxTWAIN where you can browse through them, do any manipulations you want to, save them, or delete them.

4.3 Using Direct Digital Devices


Perform the following procedure if you are using sensor images, such as Gendex Visualix HDI/eHD and KaVo Dig eXam.

- 1. Expose the IntraOral/Pan/Ceph sensor.
- 2. When the image is acquired and pre-processed, it will appear in the Image Area with all of the selected treatments applied.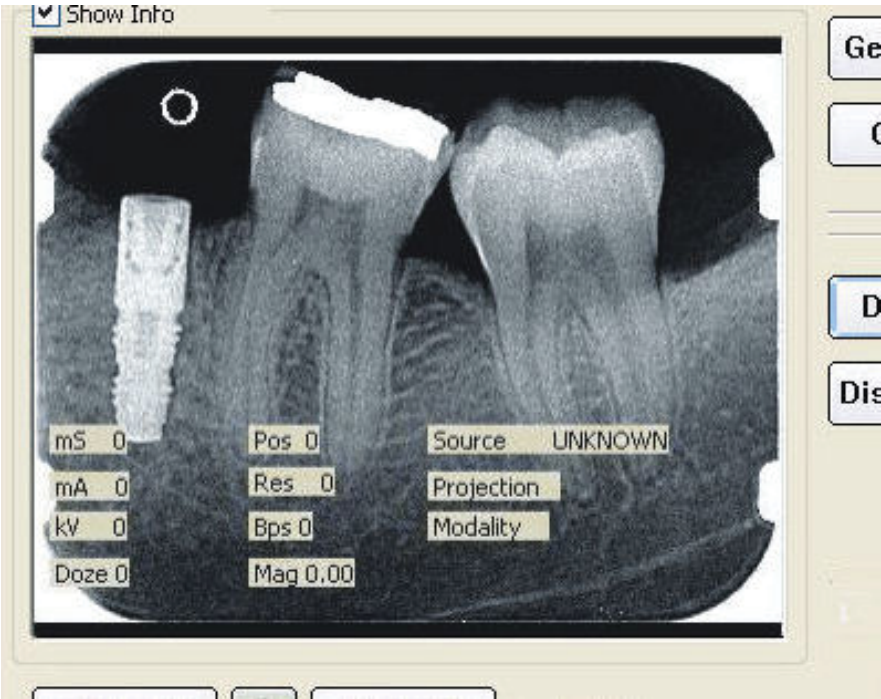

Transferring Images

Acquired or captured images appear in the Image Area of the GxTWAIN screen.

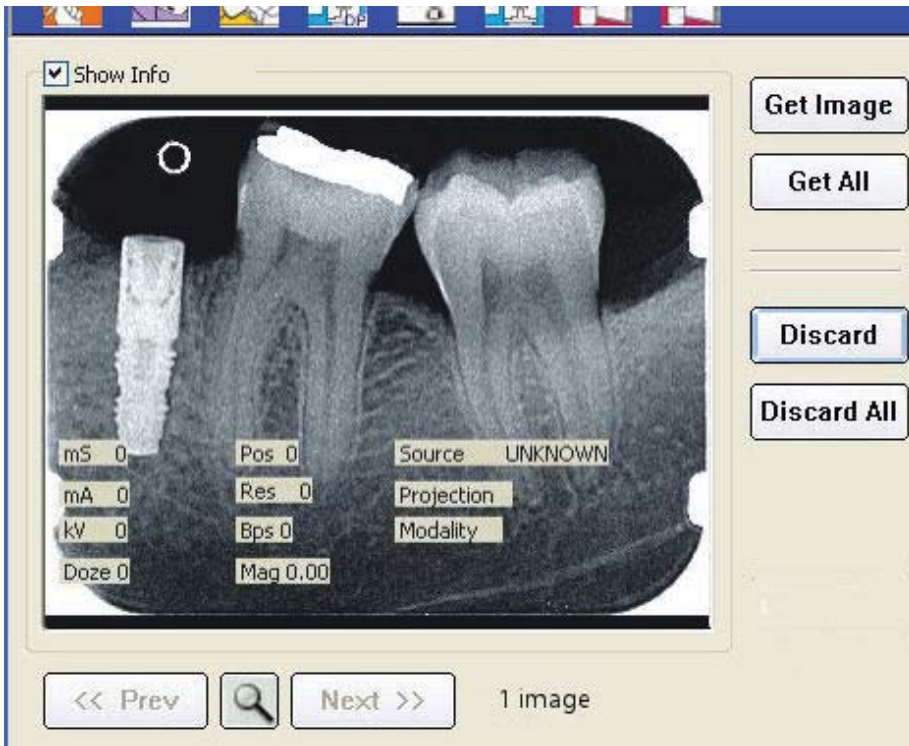The total number of images that have been captured is shown to the right of the Next button. Click **Next** or **Prev** to scroll through all of the images.

To transfer the image or images to your imaging software (ex. Photoshop, Paint Shop Pro, etc.), click **Get Image** to transfer just the current image being displayed or click **Get All** to transfer all of the images that were captured.

After transferring the image or images, you can manipulate and save the images from within your imaging software.

Note: In order to use the Get All feature, your imaging software must support multiple image transfer via TWAIN. Check the documentation for your imaging software to see if it does support this. Any images that are not transferred will remain in the temporary location they were initially saved to and will be accessible in the future. Images that are Discarded are permanently deleted and cannot be accessed again.

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Kaltenbach & Voigt GmbH