

## **How to set up the scanned image flip on the scanner LCD & PC screen.**

HOW TO SET UP THE SCANNED IMAGE FLIP ON THE SCANNER LCD & PC SCREEN

- Date : 2023.05.02
- Ver. : 1.0

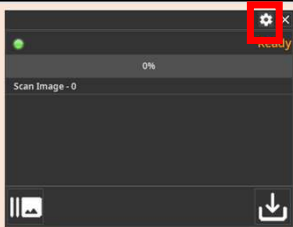
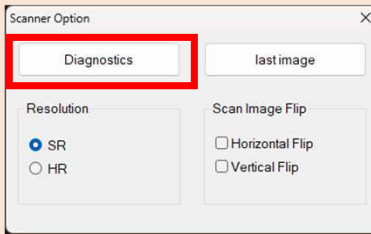
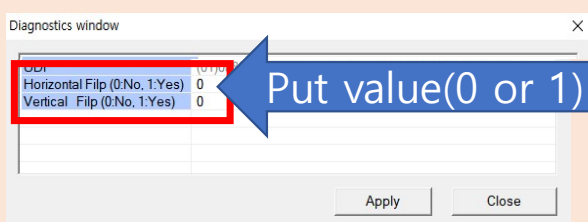
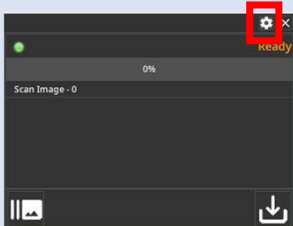
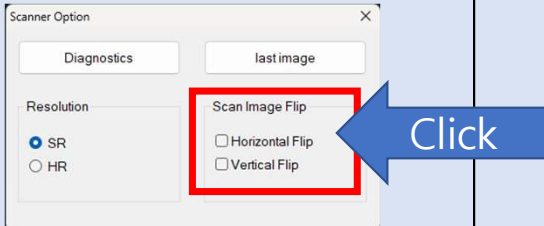


# CONTENTS

- 1 . How to set up
- 2 . Basic mode
- 3 . Example of applying horizontal flip
- 4 . Example of applying vertical flip
- 5 . Example of applying horizontal & vertical flip

## 1. How to set up

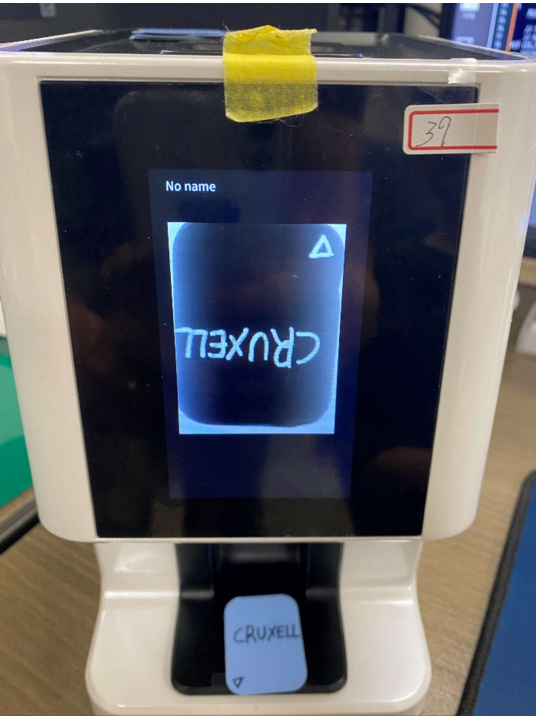
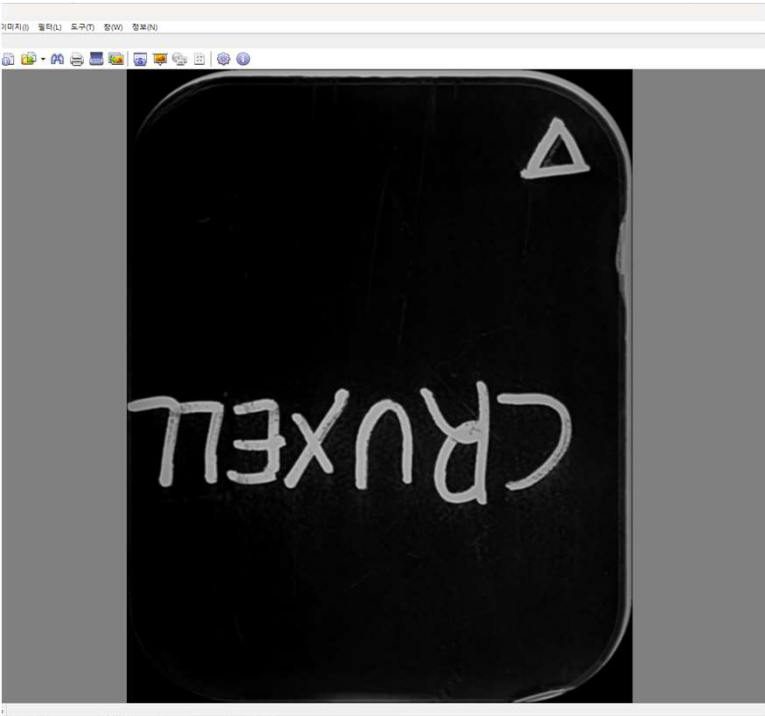
- The option setting can be used from scanner FW 180 version and SDK / TWAIN driver 1.0.2.200 version.
- The patient scan image can be flipped separately on the scanner LCD and PC screen.
- The flip function may be limited depending on the characteristics of the user viewer.

How to set up the scan image flip function			
	Step 1.	Step 2.	Step 3.
Scanner LCD Scan image flip			
PC Screen Scan image flip			


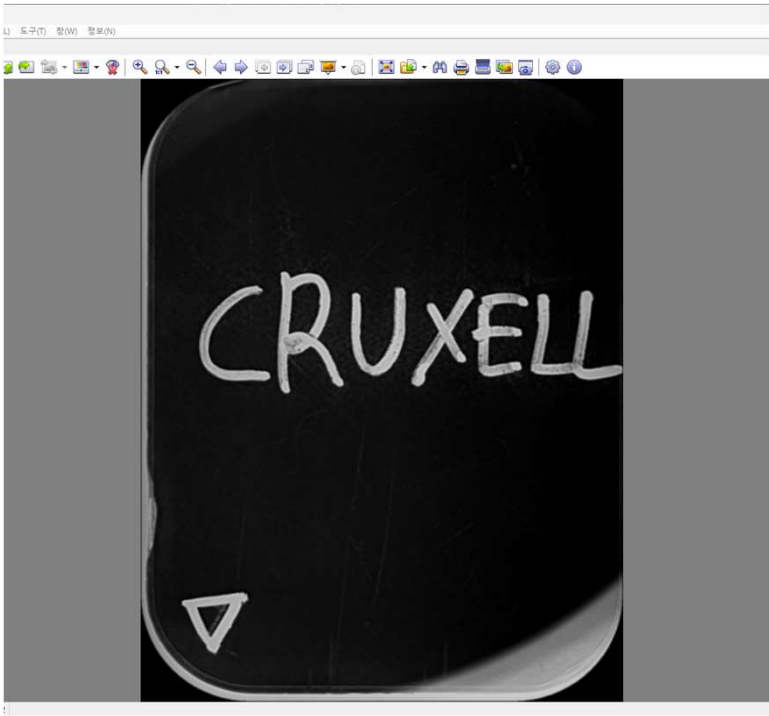
## 2. Basic mode

Basic mode	Scanner LCD	PC screen
<div><div>Horizontal Filp (0:No, 1:Yes) 0</div><div>Vertical Filp (0:No, 1:Yes) 0</div></div> <div><div><div>×</div><div>last image</div><div>Scan Image Flip</div><div><div><input type="checkbox"/> Horizontal Flip</div><div><input type="checkbox"/> Vertical Flip</div></div></div></div> <div><div><div>No name</div><div><div>△</div><div>CRUXELL</div></div></div></div> <div><div>도구(T) 정보(N)</div><div><div>△</div><div>CRUXELL</div></div></div>		

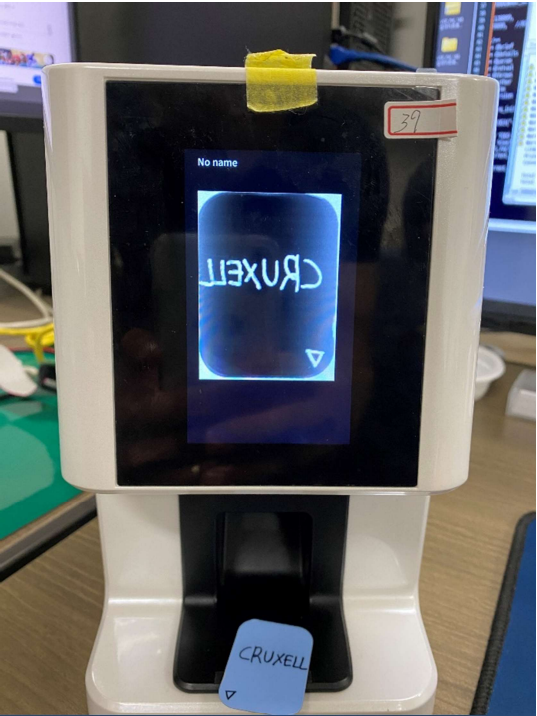

### 3. Example of applying horizontal flip

Horizontal Flip mode	Scanner LCD	PC screen
<div data-bbox="152 683 577 772"><p>Horizontal Flip (0:No, 1:Yes) <b>1</b></p><p>Vertical Flip (0:No, 1:Yes) 0</p></div> <div data-bbox="201 842 528 1056"><p>Scan Image Flip</p><p><input checked="" type="checkbox"/> Horizontal Flip</p><p><input type="checkbox"/> Vertical Flip</p></div>		

## 4. Example of applying vertical flip

Verical Flip mode	Scanner LCD	PC screen
<div data-bbox="152 691 573 774"><p>Horizontal Filp (0:No, 1:Yes) 0</p><p>Vertical Filp (0:No, 1:Yes) 1</p></div> <div data-bbox="201 845 539 1067"><p>Scan Image Flip</p><p><input type="checkbox"/> Horizontal Flip</p><p><input checked="" type="checkbox"/> Vertical Flip</p></div>		

## 5. Example of applying horizontal & vertical flip

Horizontal & Vertical Flip mode	Scanner LCD	PC screen
<p data-bbox="152 699 571 766">Horizontal Flip (0:No, 1:Yes) <b>1</b> Vertical Flip (0:No, 1:Yes) <b>1</b></p> <div data-bbox="201 845 542 1066"><p>Scan Image Flip</p><p><input checked="" type="checkbox"/> Horizontal Flip</p><p><input checked="" type="checkbox"/> Vertical Flip</p></div>	 <p>The image shows a white dental scanner with a black LCD screen. The screen displays the word 'CRUXELL' in white, which is upside down and mirrored. A yellow sticky note with the number '39' is attached to the top of the scanner. A blue label with 'CRUXELL' is attached to the base of the scanner.</p>	 <p>The image shows a PC screen displaying a software interface. The main window shows a black background with the word 'CRUXELL' in white, which is upside down and mirrored. The interface includes a menu bar at the top with options like '도구(T)', '참(0)', and '정보(N)'. A toolbar with various icons is visible below the menu bar. The status bar at the bottom shows the number '232'.</p>